

BlueLavage[®]

The environmentally friendly lavage system for orthopedics and trauma surgery



Conserving resources



65% less CO₂*



Made in Europe



german
ecodesign
award

EFFIZIENZPREIS
NRW



EUROPÄISCHE UNION
Investition in unsere Zukunft
Europäischer Fonds
für regionale Entwicklung

*than disposable systems




The next generation of lavage systems

The BlueLavage® is a completely new sterile single-use lavage system that is widely used in orthopaedics and trauma surgery.

There are no electronic components in the BlueLavage® handpiece. The system is driven by a separate power device, the so-called "DriveUnit". The DriveUnit is a drive unit that can be reused. After reaching its maximum service life, the DriveUnit is replaced and recycled by us. To charge the DriveUnit, the 4-way charger supplied is used. The DriveUnit has a smart LED operation indicator. The LED operating display is always visible through the transparent housing cover of the BlueLavage® handpiece. A colour code shows the charging and operating status of the DriveUnit and indicates in good time that replacement is imminent.

By eliminating batteries and any metal parts in the BlueLavage® handpiece, tons of hazardous waste are avoided.


In the charging station provided for the DriveUnit, up to 4 DriveUnits can be charged at the same time, always within reach. The self-explanatory design makes handling the BlueLavage® safe, quick and easy.

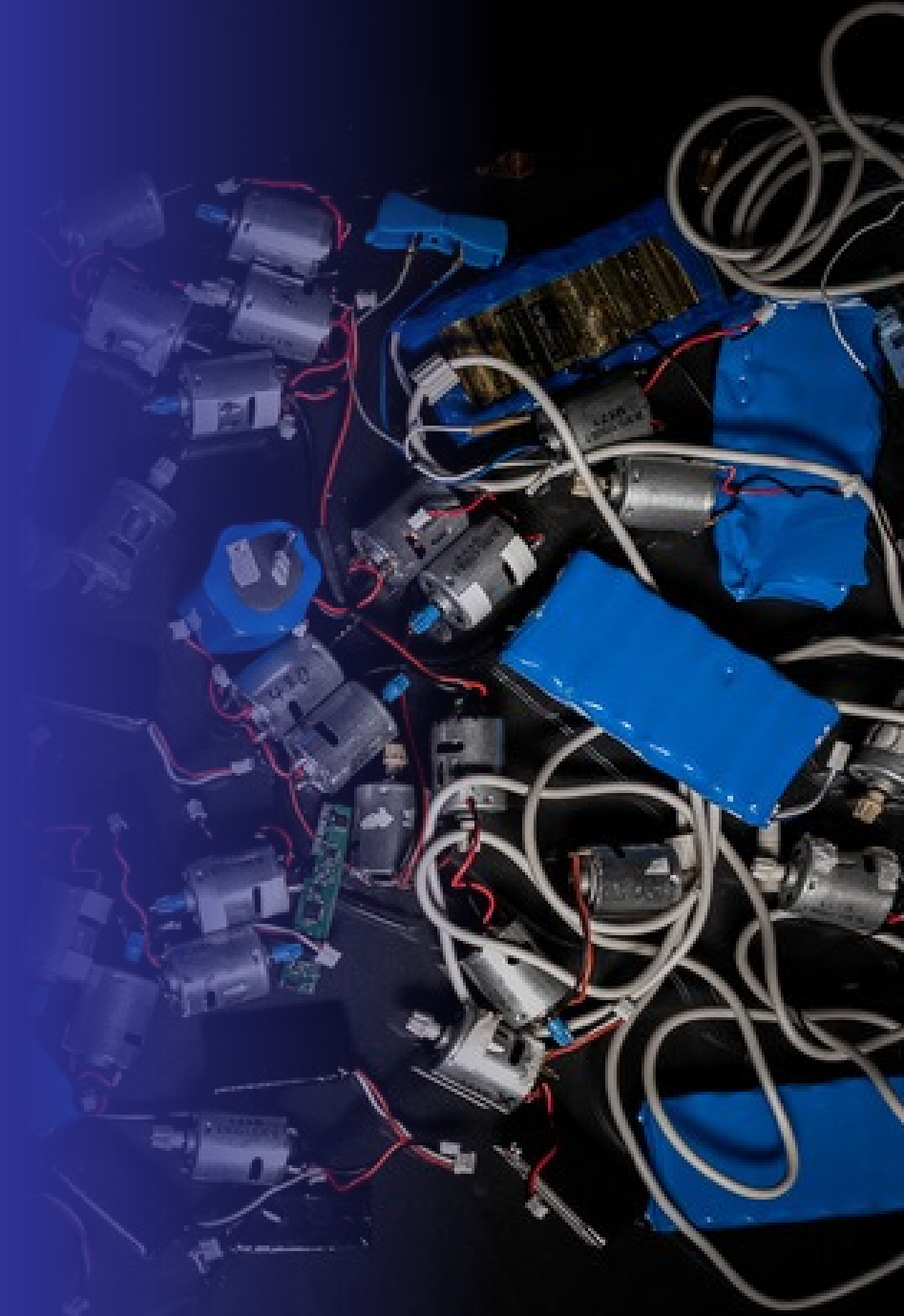

Our mission

"In the development of the product, the focus was on not using e-waste and batteries, as well as resource-efficient production, in order to take a responsible and environmentally friendly approach."

Olaf Thiessies, CEO of UTK Solution GmbH

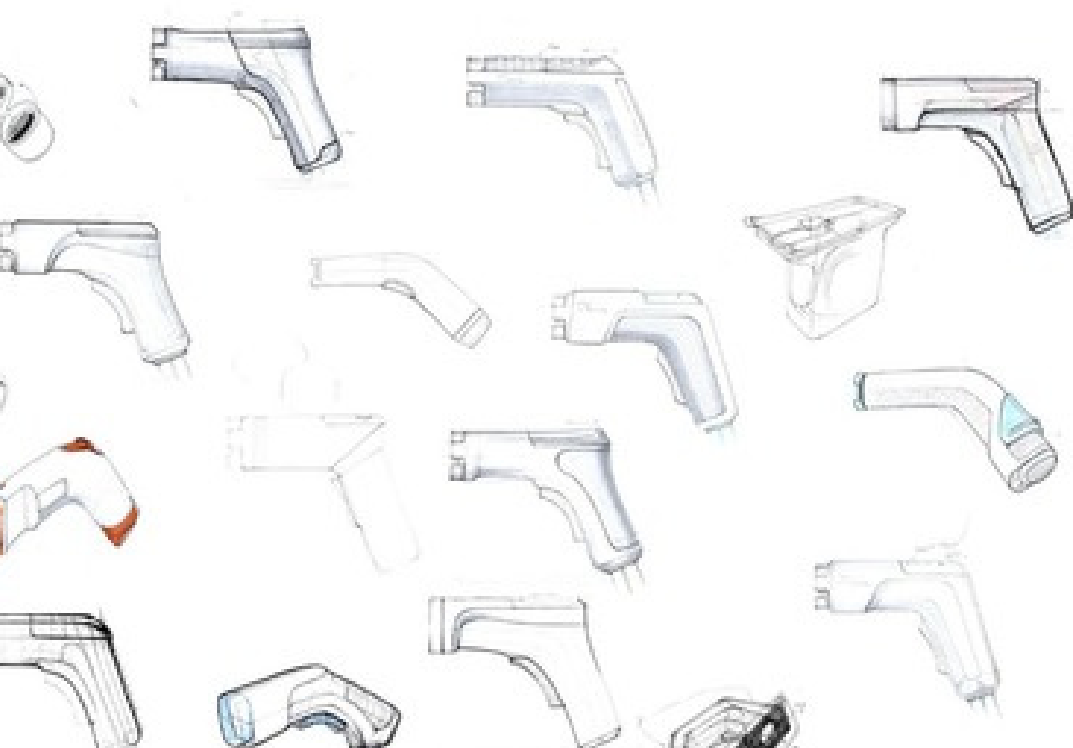

Sustainable

BlueLavage® causes 65% less CO₂* and no electronic waste.



Quality

The BlueLavage® is a high-quality medical product.



Cost-effective

BlueLavage® is fairly priced and reduces hospital disposal costs.



Available for delivery


The BlueLavage® is always ready for delivery!

*than conventional disposable systems


BlueLavage®...climate-friendly and unobtrusive sound

Resource efficiency is no coincidence.

The BlueLavage® system produces 65% less harmful CO₂ than comparable products*.

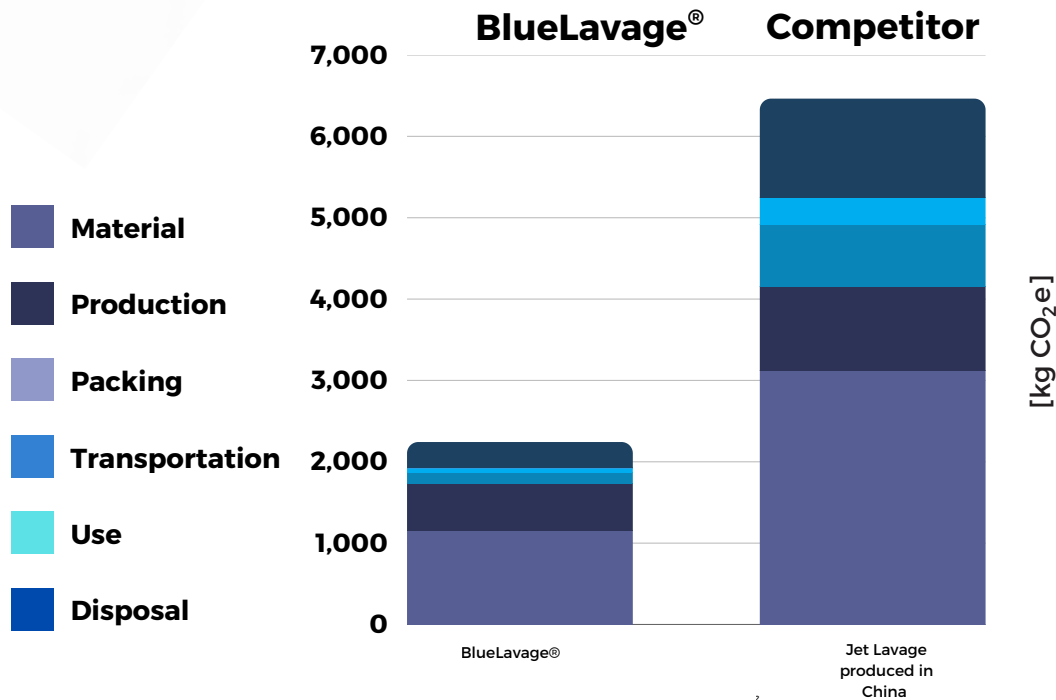

Volume measured with small and large rinse step.
 Very good performance without noise pollution
 The BlueLavage® system convinces with a sonorous sound that is clearly quieter than comparable products*.

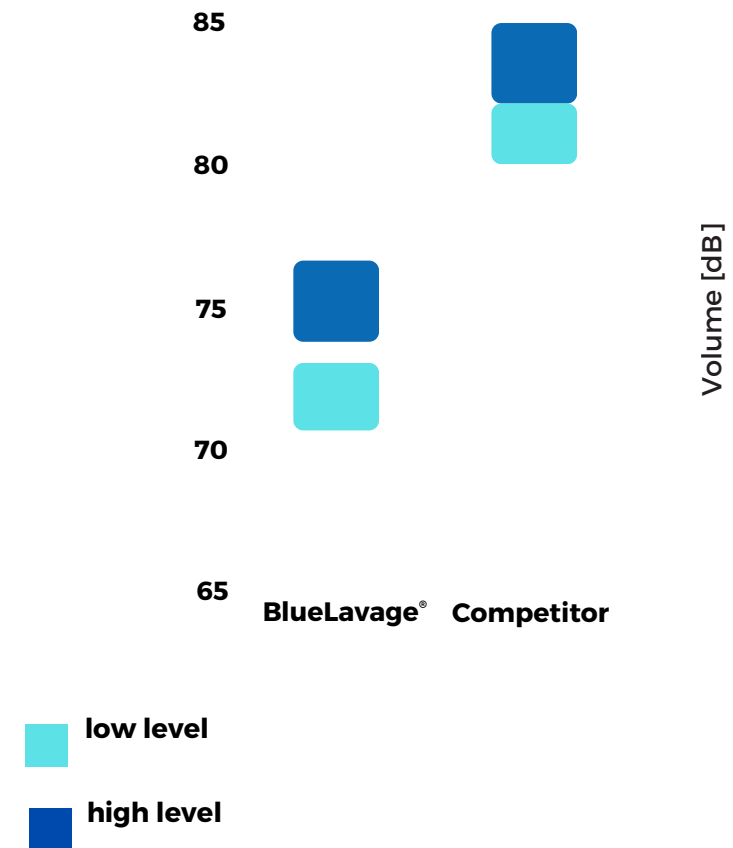

The environmentally friendly, innovative and powerful BlueLavage® rinsing system

Powerful

- Two rinse levels selectable
- HighSpeed level (Flow rate over 1,200 ml/min)
- LowSpeed level (Flow rate of 600 ml/min)
- Constant flushing pressure of 1 bar (15 psi)

Environmentally friendly

- The BlueLavage® system produces no electronic waste, no battery waste and produces 65% less harmful CO₂ than comparable products.
- Replacement and recycling of the drive unit by UTK Solution
- Tubing system without harmful or banned plasticisers
- Only recyclable and biocompatible plastics are used, which are tested according to DIN EN 10993-5

Awarded

- Our manufacturer, UTK-Solution GmbH from Lüdenscheid, was awarded the "NRW Efficiency Prize" in 2019 for the development of the BlueLavage® system.

Innovative


- Operated by a rechargeable DriveUnit
- Electronically controlled engines
- Reduced noise development
- Sensitive 2-step switch with delayed start option
- During operation, the smart LED light informs you of the current operating state.
- Sensor-based monitoring of the most important parameters
- Patent-protected locking of the drive unit in the BlueLavage® housing

User-friendly

- Thanks to the new double Tyvek® packaging, the BlueLavage® is easy to open and to take, even under sterile conditions.
- Easy Top Loading of the drive unit (DriveUnit)
- Easy and safe changing of the different irrigation attachments
- Perfect handling due to the very low weight and the absence of disturbing electric cables or compressed air hoses
- Quiet and low-vibration operation
- With the BlueLavage® you are independent of the CSSD and there are no costs for reprocessing and sterilisation



BlueLavage® rinsing system in detail

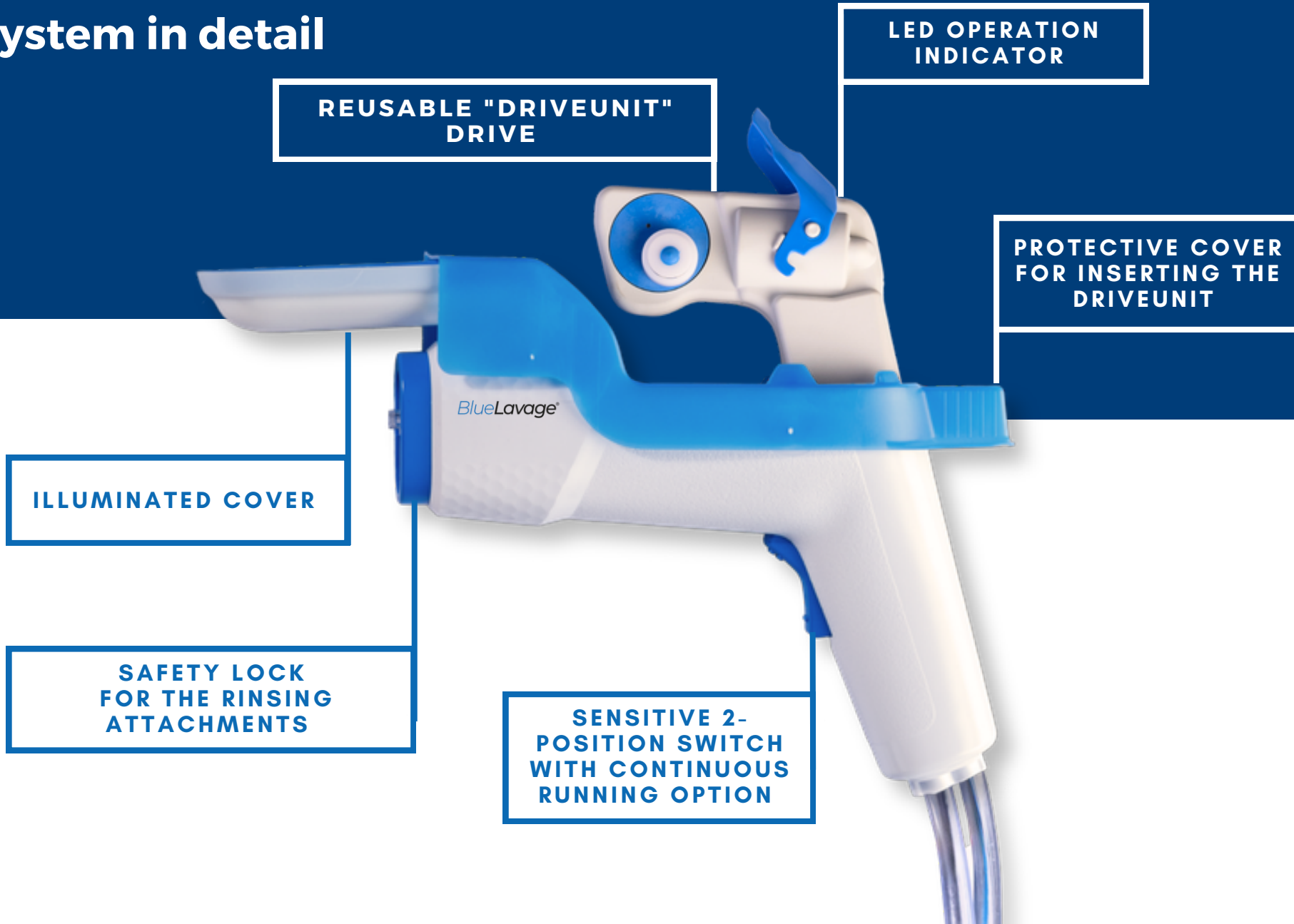

BlueLavage® DriveUnit and DockingStation

Electronic key figures DriveUnit

- Lithium-polymer battery
- Electronically regulated motor control
- During operation, there is permanent electronic monitoring of important parameters by sensors
- High capacity of the DriveUnit for an optimum cleaning result
- The DriveUnit is exchanged and recycled

Electronic key figures DockingStation

- Charging cradle for simultaneous charging of 4 DriveUnits
- Power supply 100 to 250 V AC
- Dimensions: Width 165 mm, height 55 mm, depth 146 mm

Safety Information BlueLavage® DockingStation - DriveUnit

- The BlueLavage® DockingStation and DriveUnit are marketed non-sterile and must not be sterilised.
- The responsibility for proper cleaning/disinfection of the BlueLavage® DockingStation and DriveUnit lies with the operator and product user.
- A mechanical reprocessing procedure (cleaning, disinfection, etc.) must not be used; only wipe disinfection* is permitted.

*Disinfect the DockingStation/DriveUnit using non-acidic, alkaline hygienic cleaners in accordance with the chapter Reprocessing in the instructions for use.



Meaning of the LED signals

DriveUnit light signals during usage



Steady green light

The lithium polymer battery is 100% charged.



Steady yellow light

After 1 minute, the LED light signal switches from a permanent green light to a permanent yellow light. The yellow continuous light signals that the DriveUnit is still ready for use, but that it should be fully recharged before the next operational use.



Flashing yellow quickly

The lithium polymer battery is completely discharged and not ready for use.



Steady red light

The DriveUnit has reached the end of its service life or is defective. Contact us for replacement.

Light signals while charging the DriveUnit



Steady green light

The lithium polymer battery is 100% charged.



Slowly flashing yellow light

Charging process active.



Flashing blue light

Charging process active, but the DriveUnit needs to be renewed soon.



Steady red light


The DriveUnit has reached the end of its service life or is defective. Contact us for replacement.



No light signal

The DockingStation is without power supply or the DockingStation or DriveUnit are defective.

Inserting the DriveUnit and rinsing attachments

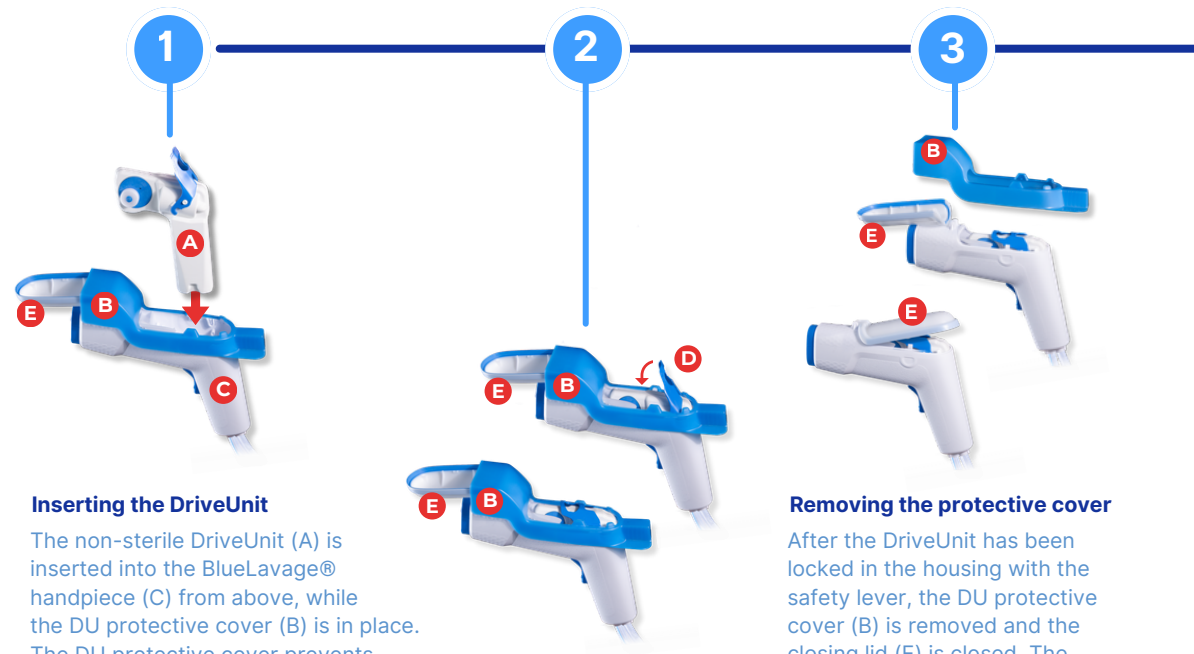

Inserting the DriveUnit

The non-sterile DriveUnit (A) is inserted into the BlueLavage® handpiece (C) from above, while the DU protective cover (B) is in place. The DU protective cover prevents accidental contact with the sterile BlueLavage® handpiece and is removed by the circulating nurse.


Securing the DriveUnit

As soon as the DriveUnit is completely inserted in the BlueLavage® handpiece, the safety lever (D) must be pushed down by the circulating nurse, so that the DriveUnit is securely locked in place.

Removing the protective cover

After the DriveUnit has been locked in the housing with the safety lever, the DU protective cover (B) is removed and the closing lid (E) is closed. The closing lid is securely fixed in its closed position by internal locking clips.

Inserting the DriveUnit and rinsing attachments

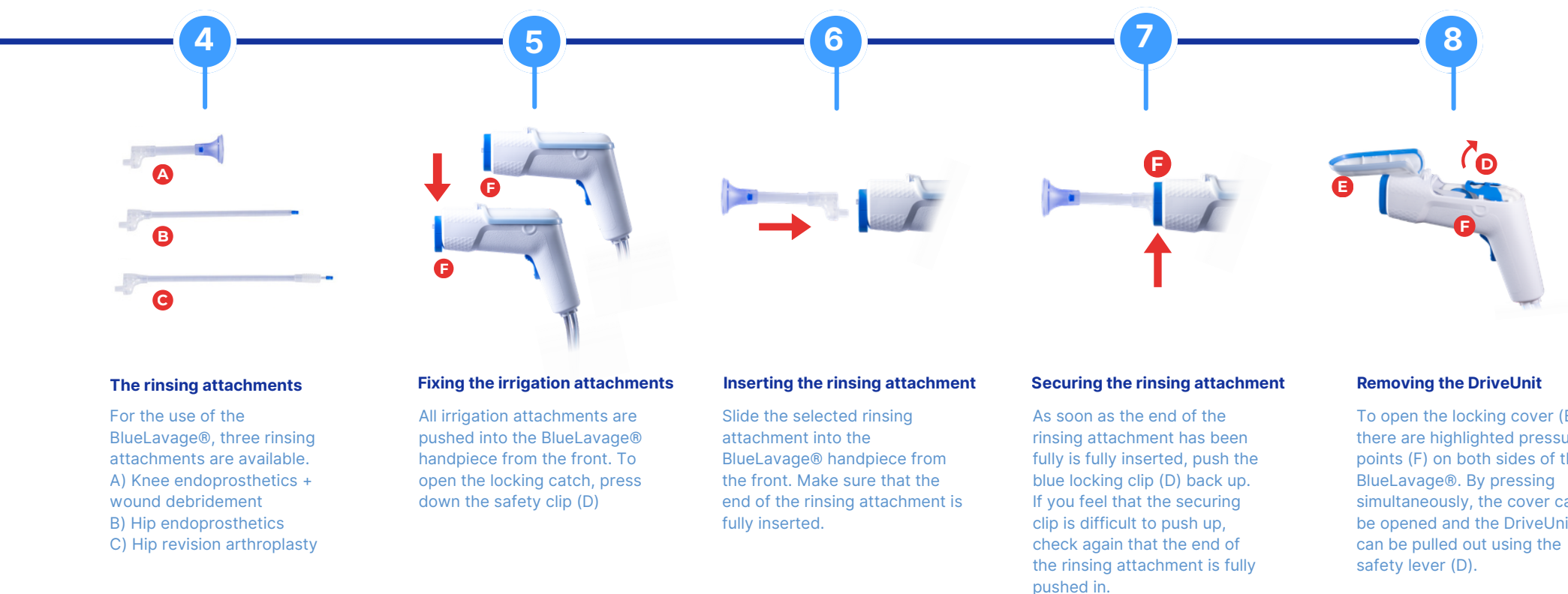

IMPORTANT NOTES ON STERILITY!

The DriveUnit is non-sterile and should never be touched by the sterile OR-Team member.

The removal of the protective cover after inserting the DriveUnit may only be removed by the circulating nurse.

Never press the lateral pressure points during operation to prevent the safety cover from accidentally "popping open".

The DriveUnit must be fully recharged after each use before it can be used again.

Sales

Sustainability shapes not only our products, but also our sales strategy. To ensure you receive first-class customer service, we have dedicated sales teams in your region and country. This is made possible through our collaboration with distributors and sales representatives.

Are you a hospital and interested in our products?

Visit our website to find the right contact for you. You can contact us, UTK Solution GmbH, directly via our contact form or UTK Solution contact details.

Are you a distributor or working as a sales representative and interested in distributing BlueLavage?

Do not hesitate to contact us. We are always interested in expanding our distribution channel.



UTK Solution GmbH
Sedanstraße 16
58507 Lüdenscheid
Germany

Phone: 02351 985241 0
Fax: 02351 985241 49
E-Mail: office@utk-solution.com



Article overview with order numbers

item number	description	Packaging unit	Scope of delivery	
BL2000.1	BlueLavage® handpiece with funnel and Funnel-TIP Disposable system, sterile	20x	1 x 2.5 meter inlet hose for rinsing liquid 1 x 2.5 meter drain hose with clip for vacuum pump 1 x short irrigation attachment with funnel for knee endoprosthetics	

Optional articles

BL2006.1	BlueLavage® <u>FunnelTip</u> , sterile, length 130 mm, opening Ø 40 mm, funnel irrigation tip	10x	Pack of 10, sterile, individually removable	
BL2004.1	BlueLavage® <u>LongBoneTip</u> , sterile, length 240 mm, femoral irrigation tip	20x	Pack of 20, sterile, individually removable	
BL2003.1	BlueLavage® <u>BrushTip</u> , sterile, total length 300 mm, including 60 mm brush, femoral canal irrigation tip with brush	20x	Pack of 20, sterile, individually removable	
BL2005.1	BlueLavage® DU protective cover, sterile, for safe insertion of the DriveUnits	5x	Pack of 5, sterile, individually removable	

Investment material

BL2001.2	BlueLavage® <u>DriveUnit</u>	2x	Packing with 2 single DriveUnits, for continuous use	
BL2002.02	BlueLavage® <u>DockingStation</u> Charging station with power supply and adapter 1 x 4-compartment charging station	1x	1 x Power supply unit with 10 V output 1 x Country specific adapter for the power supply unit	



Disclaimer

All rights reserved. Reprints, including excerpts, digital and reprographic reproductions are only permitted with the written consent of UTK Solution GmbH, Sedanstraße 16, 58507 Lüdenscheid, Germany. Violations will be legally punished. Our "General Terms and Conditions of Sale and Delivery" apply exclusively to all offers, sales and deliveries. Program and design changes as well as deviations of the actual designs from the illustrations are reserved.

Manufacturer:

UTK Solution GmbH
Sedanstraße 16
58507 Lüdenscheid
Germany

Distributed by:

UTK Solution GmbH
Sedanstraße 16
58507 Lüdenscheid
Germany

Phone: + 49 (0) 2351 9852410
E-Mail: office@utk-solution.com

Phone: + 49 (0) 2351 9852410
E-Mail: office@utk-solution.com

BlueLavage-Product description-EN-UTKSolutionGmbH
Rev. 026 July 2026